Scan for info

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Orig.rev 6 2024-09-12

Dictus Original - User information ENG

Important information about using the Dictus Band

Declaration of Conformity

This AFO drop foot orthosis is a class 1 medical device that meets the applicable requirements of regulation 2017/745 MDR. Erimed stands as a guarantor for the product on the condition that the product is not handled in any other way than intended. Unapproved changes to the product make this declaration invalid.

Intended use:

The product has been developed on the basis of tests and clinical evaluations by the profession. The Dictus band is an orthotic aid primarily intended for individuals with so-called drop foot. The product is designed to help lift the forefoot using a dynamic rubber band.

The Dictus Band consists of: -Ankle strap in natural-tanned leather in two layers fitted with nylon Velcro. Plastic D-ring, latex rubber band, stainless steel attachment-hooks, plastic mono-hook.

Restrictions:

Although every effort has been made to achieve maximum compliance with function, durability and comfort, there are no guarantees that any damage will be prevented by using this product. Warranty 12 months with the exception of rubber bands for which a 3 months warranty is provided. The guarantee applies on the condition that the user has followed all instructions and care advice.

Use/Instructions for use:

Inspect the product carefully before use and do not use if any part shows obvious wear or any type of damage. Make sure the correct size is used (see size chart). The ankle strap is attached to the outside of the sock (not directly against the skin) using the Velcro straps. Two fastening hooks are inserted into lace holes (eyelets) in the shoe. Alternative mono-hook is mounted between tongue and lacing. The rubber band is hooked through the D-ring on the ankle strap, and the seam of the rubber band should be placed in the middle of the knot at the D-ring. Other placement can lead to the rubber band breaking during otherwise normal use. The rubber band is then hooked onto the steel hooks (or mono-hook). The lifting force of the rubber band is determined by the distance between the D-ring and the attachment hook/hooks. Always try out the appropriate lacing hole/distance for the hooks so that the intended function is achieved. At each use, check that the fasteners in the ankle straps and shoes are correctly fitted, and that all parts are intact. The rubber band must be changed when necessary, and at least every 3 months or more often with extensive use. If the rubber band should break or one of the fastenings should come loose, the foot will no longer be held up, and there is a risk of the user tripping or falling. A spare rubber band should always be at hand.

Cleaning and maintenance of the product:

- *For leather – use a soft damp cloth and possibly leather cream.
- *Close the Velcro straps when the product is not in use. Make sure that particles or hairs are not caught in the velcro.
- *Avoid aggressive liquids (bleach, ammonia, solvents, etc.) and abrasive materials (e.g. brushes, steel wool, etc.) to remove stains.
- *Only gentle hand wash in minimum 30°C.
- *Let the product air dry and never use heat sources as a drying aid.
- *Stored dry, cool and dark, avoid heat and humid environment.
- *Note that sunlight reduces the lifespan of latex rubber (rubber bands).

Waste disposal

The product must be disposed of according to local regulations.

Other information

We recommend that the product is prescribed and adjusted by the healthcare professional and instructions must be given in such a way that the user can safely understand this information. The user information is important and must be saved for the entire life of the product. Serious incidents with the product must be reported to the manufacturer and the competent authority of the Member State where the user is established.

Limitations of liability and improper use.

- Production is done according to the best available technology. Erimed assumes no responsibility if the product is used under incorrect conditions or is changed/handled as described below.
- *lack of compliance with the manufacturer's instructions and safety regulations, or use contrary to these.
- *use for other than the intended use of the product.
- *cleaning with unsuitable products and unsuitable storage.
- *use by people with allergies to the manufacturing material.
- *The product is not flame-proof or water-resistant.
- *The packaging contains small parts, and the product is not a toy.

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